

# On Call® Sure Platform

Smart Diabetes Management starts with On Call®

**On·Call®**

## Results you can trust from a meter designed for performance.

Built with industry leading specs, the On Call Sure platform is equipped to **handle the demands** of professional channels.



**Clinically Proven Accuracy**



**FAD-GDH Enzyme**

Advantage of venous and arterial testing



**Wide Hematocrit Range**



**INTEGRATED BLUETOOTH**  
CONSULT YOUR LOCAL  
DISTRIBUTOR FOR CURRENT  
MOBILE APP COMPATIBILITY


## On Call Sure and Sure Sync

### SPECIFICATIONS & FEATURES

- On Call® Sure and On Call® Sure Sync use the same strip technology
- 0.6 uL Sample Size
- Results in 5 seconds
- 10 – 70% HCT Range
- FAD-GDH Enzyme offers advantage of venous blood testing and is validated for neonatal testing
- Auto-coding
- Active HCT Correction
- Individually foil wrapped strips
- Large, easy-view display

## Clinically Proven Accuracy

Equivalent correlation to the reference method provides evidence of high accurate results with capillary blood samples from fingertip.

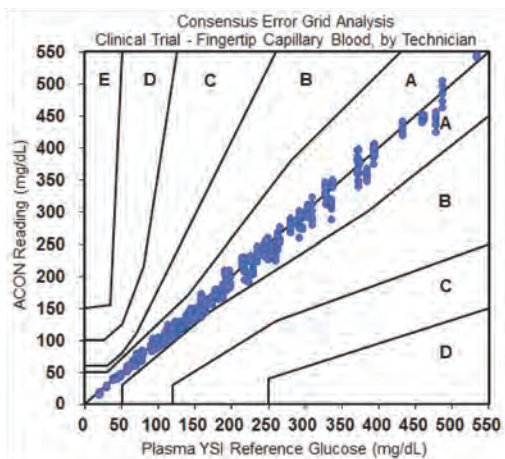

Finger Sample Site Tested by Technician			
System Accuracy Results for Glucose Concentration <5.55 mmol/L (<100 mg/dL)			
Within ±0.28 mmol/L (Within ±5 mg/dL)	Within ±0.56 mmol/L (Within ±10 mg/dL)	Within ±0.83 mmol/L (Within ±15 mg/dL)	
156 / 216 ( 72.2% )	214 / 216 ( 99.1% )	216 / 216 ( 100.0% )	
System Accuracy Results for Glucose Concentration ≥5.55 mmol/L (≥100 mg/dL)			
Within ±5%	Within ±10%	Within ±12%	Within ±15%
335 / 582 ( 57.6% )	530 / 582 ( 91.1% )	566 / 582 ( 97.3% )	582 / 582 ( 100.0% )
System Accuracy Results for Glucose Concentrations Between 1.09mmol/L (19.6mg/dL) and 29.6mmol/L (533.5mg/dL)			
Within ±0.83mmol/L (±15mg/dL) or ±15%			
798 / 798 ( 100% )			
Within ±0.56mmol/L (±10mg/dL) or ±12%			
780 / 798 ( 97.7% )			

\*Venous, Arterial and Neonatal results are available upon request.

## Ordering Information

PRODUCT NAME	CATALOG NO.	CONTENTS
On Call <sup>®</sup> Sure Blood Glucose Monitoring System	G115-10U	1 meter, 10 strips, 1 lancing device, 10 lancets, 1 control
	G115-11U	1 meter, 1 lancing device
On Call <sup>®</sup> Sure Blood Glucose Meter	G115-12U	1 meter, 1 control, 1 lancing device, 10 lancets
	G115-13U	1 meter, 1 control
On Call <sup>®</sup> Sure Sync Blood Glucose Monitoring System	G115-10W	1 meter, 10 strips, 1 lancing device, 10 lancets, 1 control
	G115-11W	1 meter, 1 lancing device
On Call <sup>®</sup> Sure Sync Blood Glucose Meter	G115-12W	1 meter, 1 control, 1 lancing device, 10 lancets
	G115-13W	1 meter, 1 control
	On Call <sup>®</sup> Sure Test Strips	G135-10U
50 Test Strips (25/vial, 2 vials/kit)		
100 Test Strips (25/vial, 4 vials/kit)		
50 Test Strips (50/vial, 1 vial/kit)		
G135-11U		100 Test Strips (50/vial, 2 vials/kit)
	25 Test Strips (pouched packaging, 25 pouches/kit)	
		50 Test Strips (pouched packaging, 50 pouches/kit)
On Call <sup>®</sup> Sure Control Solution	G125-12U	Control Solution (level 0, level 1, level 2)
On Call <sup>®</sup> Lancets	G124-10A	100 Lancets (25/bag)
On Call <sup>®</sup> Lancing Device	G124-11A	1 Lancing Device



aconlabs.com

ACON Laboratories, Inc.  
10125 Mesa Rim Road, San Diego, CA 92121, U.S.A.  
Tel: 1.858.875.8000  
Fax: 1.858.200.0729  
Email: info@aconlabs.com